Attachment 1-2 Rev. "06" of GA-CopiOs Membrane CE-E

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Contraindications

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- Absorption and tissue regeneration processes begin after one or two days and may last for weeks or even months.

Interactions

None known

Magnetic Resonance (MR) environment: As the device is composed of natural collagen, it is safe for magnetic resonance environment. No MR artefacts are expected due to the composition of the material.

Adverse reactions

Local reactions may rarely occur in patients who are not known to be hypersensitive to foreign collagen; however, these reactions resolve rapidly. Patients should be told to inform the doctor about any possible side effects. All adverse reactions must be reported to Tutogen Medical GmbH.

Shelf life

Do not use the CopiOs Pericardium Membrane after the expiry date (📅) printed on the inner pouch and cardboard box.

Storage instructions

Store this medical device at room temperature between 15 °C and 30 °C, in a clean, dry place; protected from direct sunlight.

Date of last revision

09/2017 "06"

CopiOs® is a registered trademark of Zimmer Biomet or its affiliates.

Tutoplast® is a registered trademark of Tutogen Medical GmbH.

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Do not reuse	Manufacturer	Keep away from direct sunlight
Do not re-sterilise	Sterilised by irradiation	Keep in a dry place
Batch number	Use by	Storage temperature
Order number	Read the instructions for use	Do not use if the packaging is damaged

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Tutoplast® es una marca comercial registrada de Tutogen Medical GmbH.